USAGE OF FFP2

1
Unfold the mask and pull the ear-loop bands outward on either end

2
Position the mask on your face with adjustable metal nose piece facing up

3
Pull the ear-loop bands and place them around both ears

4
Press down to mold the metal nose piece ensuring a tight-fitting seal

STORAGE CONDITIONS AND METHODS:

Store room temperature 10-30 C at less than 70% relative humidity and avoid direct sunlight exposure. Well ventilated and clean environment without corrosive gas. Please do not use if the product gets mildewed or deteriorated due to improper storage

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INSTRUCTIONS

Standard:

This product meets the EN 149:2001:A1+2009 standard for respiratory protective devices - Filtering half masks to protect against particles. This Filterin face mask is classified as category-III Personal Protective Equipment (PPE) impressed CE marking in conformity with European Commission Regulation (EU) 2016/425 regarding to PPE

Intended Use:

The dust mask is classified as category FFP2. It provides protection against particulates, mist, smoke and oil based aerosols. The packaging provides the mask with hygienic protection prior to its use. Can effectively protect pollen allergy, etc. The mask can only be used for professional protection, not for medical purposes.

Warnings:

The user must be trained and given the correct instructions about how to wear the mask. FFP masks do not provide protection and vapours. The ambient oxygen concentration may not be less than 19.6 Vol.%. The masks should not be deployed if the concentration, type or nature of the contamination is unknown or is of a hazardous level.

Masks must be replaced after they have been damaged, if the breathing resistance is increased as a result of clogging, and after use.

No changes may be made to the mask.

Instructions:

The mask is intended to be used only once. Do not use the mask more than once.

Keep the packaging closed until you will use the mask.

Replace the mask if it is damaged or if the breathing resistance increases due to clogging.

Not to be used during firefighting activities.

Not to be used in explosive atmosphere.

Non-observance of the instructions and warnings for using this mask could lead to a poorer performance of the mask and could result in health problems.

Keep facial hair underneath the mask. The mask can not offer maximum protection when it fails to properly connect to your face due to facial hair.

Transport and Storage:

Do not expose to direct sunlight. Store in a dry and frost-free location.

Usage:

Please follow the diagram for proper usage.

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